

Instant Cold Compress



SHAKE



COMPRESS



SHAKE



APPLY

To relieve pain and inflammation caused by nerve and muscles stresses, bruises and minor burns.

For best results, store the compress between 32 and 80 degrees Fahrenheit (0-27° C).

Directions: The compress should be wrapped in a towel or piece of cloth to prevent discomfort and damage to the skin.

Warning: If the compress ruptures, the fluid within should not have contact with the skin or eyes: if this happens, rinse immediately with cold water. The contents should not be ingested. If this happens accidentally call the hospital (or poison control center) immediately. Keep out of the reach of children.

**For external use only.
For single use only.**

Do not puncture the plastic surface of the compress.

CONTAINS: Urea and Water

Manufactured in China for Urgent First Aid.



www.UrgentFirstAid.com

Consulte el otro lado para español



FDA



ISO 9001 2000

REORDER # M563-E